# Personal Scale

# HOME-USE PERSONAL SCALE USER'S MANUAL SYE-2004A8

# Specification

Capacity: 180kg/396lb

• Division: 0.1kg/0.2lb

Available units: kg/lb

Display for low battery power: "LO"

Wrong indicate: "Err"

Over load indicate: "O\_Ld"

• The zero weight display "\_\_\_\_"

• Power: 2 \*CR2032 lithium batteries

Foot tap start

#### Instruction

- Open the battery cover of the scale and set the unit to the preferred measurement kg or lb.
- Put the scale on a solid and flat floor and make sure the surface of the scale is dry to avoid slip.
- Tap slightly on the glass of the scale and the display show "0.0".
- Stand calmly on the scale when "0.0" appears on the LCD display with both feet share the average weight.
- When weighing, the read-out rises and display flashes till determined.
- The unit "kg/lb" stops flashing and display your weight read-out.
  Attention: never jump onto the scale to avoid damage.

## **Warning Indicate**

- Lo Replace the batteries if "Lo" appears in the display window.
- Err The scale is unable to be used if "Err" appears in the display window. Stand calmly onto the scale to retry after a while.
- O Ld Recorded weight is greater than maximum.

### Attention

- Results from different scale slightly differ.
- Do not put anything on the scale, unless in use to avoid unnecessary power consumption. Take the batteries out when the scale not being used for a long period of time.
- Keep the scale in a clean and dry environment.
- Your glass scale represents a sensitive electronic instrument. So high-frequency devices may cause disturbance when used in the immediate vicinity. If faults caused in this way, please keep the scale away from the source of disturbance.
- Home-use personal scale, not for commercial using.